

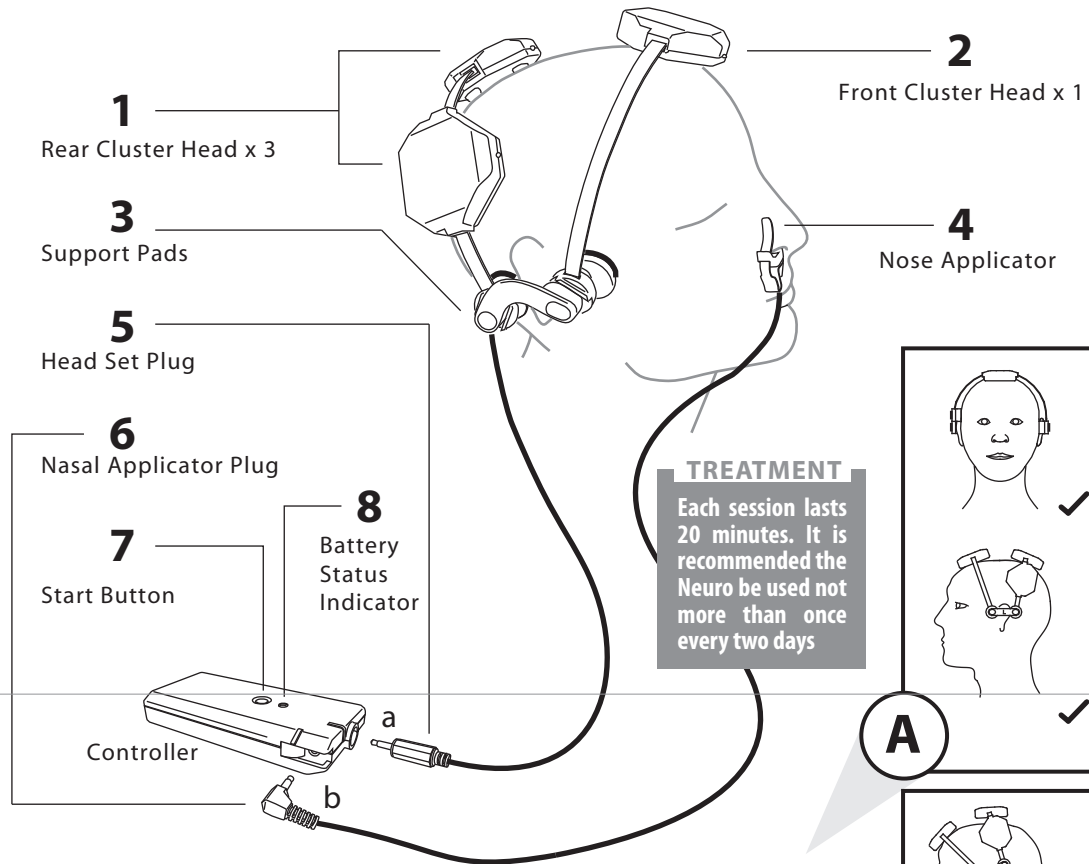


NeURO USER'S GUIDE

OPERATING INSTRUCTIONS

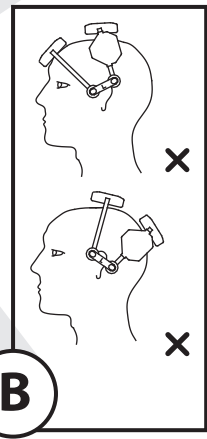


TREATMENT
Each session lasts 20 minutes. It is recommended the Neuro be used not more than once every two days

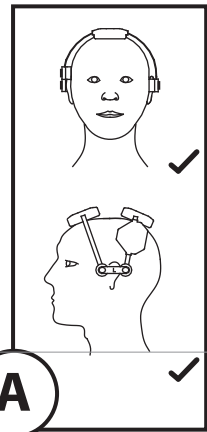


- 1 & 2** Position the rear and front cluster heads as shown in (A). Ensure that the clusters are in contact with the skin. (B) shows the cluster heads worn incorrectly in an excessive forward or backward position.
- 3** After steps 1 & 2, the support pads should naturally fall into positions around the ears. Their positions are determined by the locations of the cluster heads.
- 4** Insert the nose applicator into either the right or left nostril with the clip on the outside.
- 5** Insert the head set plug into the socket (a) of the controller.
- 6** Insert the nose applicator plug into socket (b) of the controller.
- 7** Press the start button to begin treatment. It switches off automatically after 20 minutes.
- 8** The battery status indicator light blinks when the battery power reserve is low. Battery charging instructions are on the next page.

Indicator Status

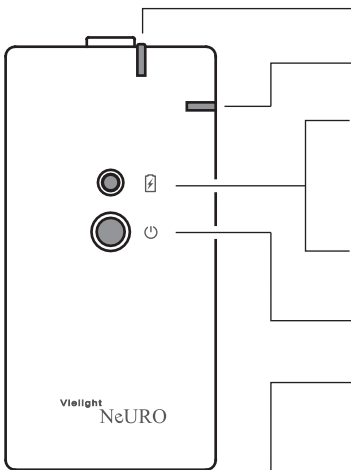

Pulsing / Red	Head set not completely plugged in or is faulty
Pulsing / Red	Nose applicator not completely plugged in or is faulty
None	Power reserve is sufficient
Pulsing / Red	Power reserve is low
Steady / Red	Charging is in progress. The light switches off automatically when charging is completed
Pulsing / Green	Controller is switched on
Red / Pulsing	Cluster head in operation

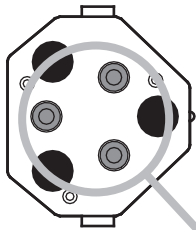


A deep red light is observable when looking directly at a functioning diode that has been switched on

Note: During treatment with the Neuro, users should not experience adverse physical sensation. However, if you experience discomfort, please contact support@vielight.com

Care & Cleaning

1. To clean the Nose Applicator, gently wipe the surface with a swab moistened with rubbing alcohol or hydrogen peroxide. You can wipe the applicator regularly with clean water.
2. Use a soft, dry cloth to clean the other surfaces. Do not use abrasive cleaners. Do not submerge any component in water.
3. Store the equipment in a dry location, free from dust, away from direct sunlight. The storage temperature should be within 10°C to 40°C.

CHARGING THE BATTERY

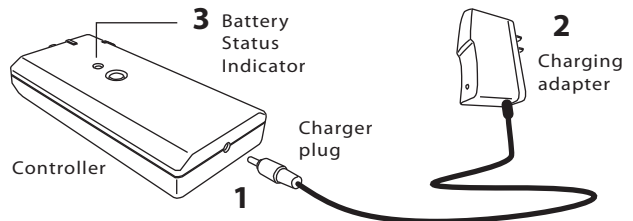

1. Insert the charger plug completely into the charging socket of the controller.
2. Connect the charging adapter to a power source.
3. The indicator light is on during charging. The light will switch off automatically when charging is complete. The maximum charging time is three hours with a flat battery.

Specification

Light Source:	Solid-state 810 nm LED
Power:	Regulated, pulsed at 10 Hz
Auto Off Timer:	20 minutes
Battery Type:	NiMH
Power Adapter:	AC Input: 100-240 V, 0.3 A, 50/60 Hz DC Output: 7.5 V, 1 A

Limited one-Year warranty

Your Vielight Neuro is warranted to be free of defect in material and workmanship under normal use.

This warranty is limited to a period of one year from the purchase date. Within this period, repairs or exchange of parts that are deemed necessary by the company will be made free of charge.

This warranty does not cover damage by the following;

- a. Misuse, abuse or alteration to the product
- b. Damage caused by accident or neglect
- c. Service or repairs made by unauthorized persons

WE SHALL NOT BE RESPONSIBLE FOR ANY INCIDENTAL, SPECIAL, OR CONSEQUENTIAL DAMAGES RESULTING FROM THE USE OF THIS PRODUCT.