BRUKSANVISNING / User Guide / Anleitung

MADRASSÖVERDRAG / Mattress Cover / Matratzenbezug Exchange 2, 3 & ST

10 cm halkskydd på varje sida / 10 cm anti-slip on each side / 10 cm rutschfest auf jeder seite

○ **MC 004-3010** [HMI: 71684]
Exchange 2 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 80 x H 19 cm

○ **MC 004-3031** [HMI: 89075]
Exchange 2 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 150 cm

○ **MC 004-3042** [HMI 102669]
Exchange 2 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 85 x H 25 cm

○ **MC 004-3012** [HMI: 81785]
Exchange 2 - **X-Large**

Mått / Dimensions / Größe:
L 200 x B/W 100 x H 19 cm

○ **MC 004-3015** [HMI: 77598]
Exchange 2 - **XX-Large**

Mått / Dimensions / Größe:
L 200 x B/W 120 x H 19 cm

○ **MC 004-3016** [HMI: 83692]
Exchange 2 - **Large NIMBUS**

Mått / Dimensions / Größe:
L 200 x B/W 80 x H 19 cm

○ **MC 004-3011** [HMI: 73522]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 80 x H 19 cm

○ **MC 004-3033** [HMI: 100089]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 85 x H 20 cm

○ **MC 004-3043** [HMI: 102670]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 85 x H 25 cm

○ **MC 004-3029** [HMI: 102336]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 90 x H 18 cm

○ **MC 004-3017** [HMI: 79779]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 95 x H 19 cm

○ **MC 004-3022** [HMI: 83693]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 210 x B/W 85 x H 19 cm

○ **MC 004-3018** [HMI: 79781]
Exchange 3 - **Large**

Mått / Dimensions / Größe:
L 210 x B/W 95 x H 19 cm

○ **MC 004-3013** [HMI: 81784]
Exchange 3 - **X-Large**

Mått / Dimensions / Größe: L 200 x
B/W 100 x H 19 cm

○ **MC 004-3044** [HMI: 105258]
Exchange 3 - **X-Large**

Mått / Dimensions / Größe:
L 200 x B/W 105 x H 20 cm

○ **MC 004-3019** [HMI: 81783]
Exchange 3 - **XX-Large**

Mått / Dimensions / Größe:
L 200 x B/W 120 x H 19 cm

○ **MC 004-3023** [HMI: 83694]
Exchange 3 - **XX-Large**

Mått / Dimensions / Größe:
L 210 x B/W 115 x H 19 cm

○ **MC 004-3021** [HMI: 82209]
Exchange 3 - **XXX-Large**

Mått / Dimensions / Größe:
L 200 x B/W 140 x H 19 cm

○ **MC 004-3024** [HMI: 83695]
Exchange 3 - **XXX-Large**

Mått / Dimensions / Größe:
L 210 x B/W 145 x H 19 cm

○ **MC 004-3090** [HMI: 89408]
NIMBUS 2

Mått / Dimensions / Größe:
L 200 x B/W 85 x H 25 cm

○ **MC 004-3039** [HMI: 89076]
Exchange ST - **Large**

Mått / Dimensions / Größe:
L 200 x B/W 80 x H 19 cm

 **master care**[®]
HUMAN HANDLING SYSTEM

PERNOVA

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Osäker?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately
- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the top piece

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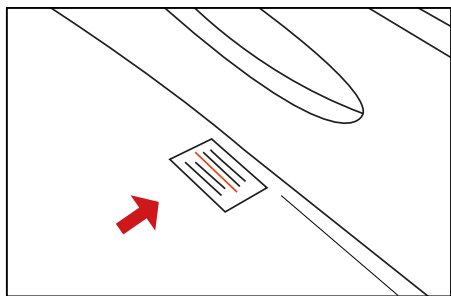
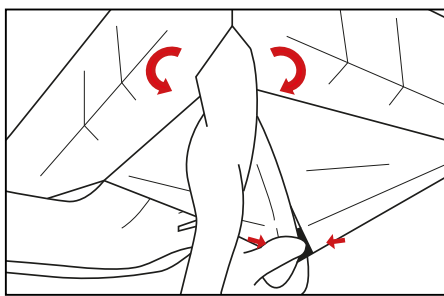
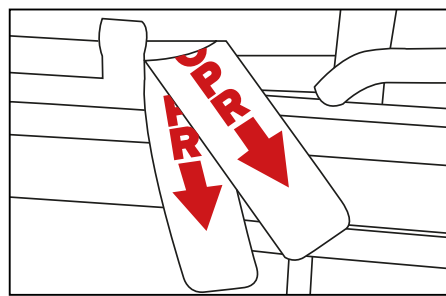
UK Place the mattress cover with the "This side toward the headrest" label by the top of the mattress and fold the cover down around its top corners

At the bottom of the cover, open the velcro strap on the same side as the air tube exit. Fit the cover around the air tube and close the velcro strap. Fold the opposite corner around the mattress

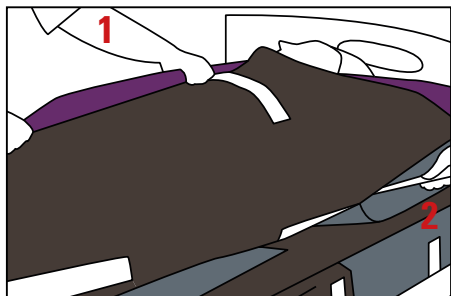
The cover has been fitted correctly when the CPR opening is visible and there is 10 cm of non-slip on both sides

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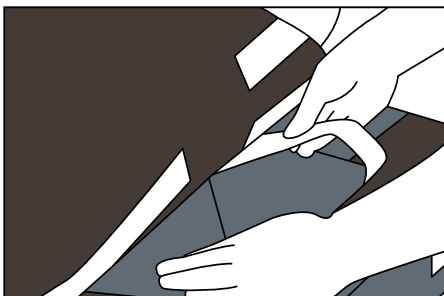
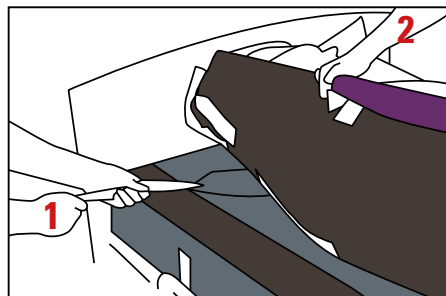
UK Person 1 or a lift helps ensure that the Master Turner fits snugly around the user. Helper 2 loosens the sliding flap's two velcro straps and folds the bottom strap over the top one

Then fold the top strap over the bottom strap and slide the top strap under the user's neck

Person 2 or a lift ensures that the Master Turner fits snugly around the user and person 1 pulls the sliding flap out under the user

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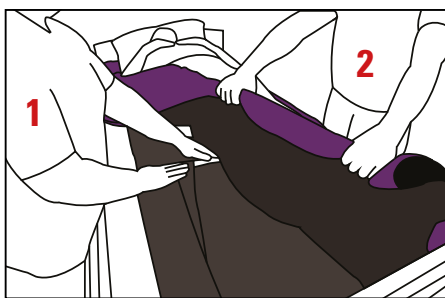
UK The Master Turner now locks against the non-slip surface on the mattress cover. The user's body is now locked against the bedding in the area where the sliding flap has been removed

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
Person 2 holds the turn sheet and helps position the user on their side. Double up the sliding flap and fold it under the user with straps at the front

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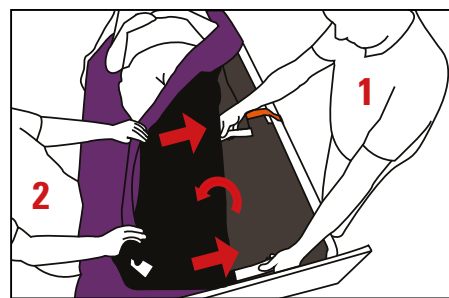
Person 2 holds the turn sheet up on the opposite side. Person 1 takes hold of the straps and pulls the sliding flap into position under the user

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
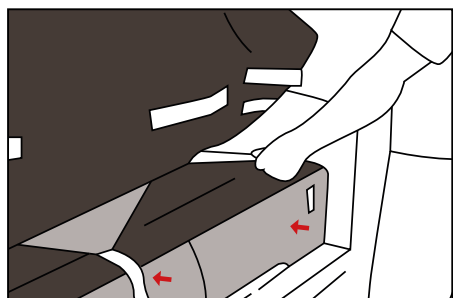
UK Fasten the straps onto the mattress cover

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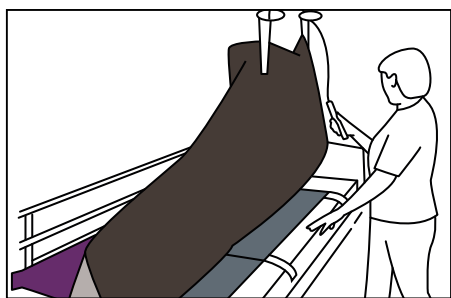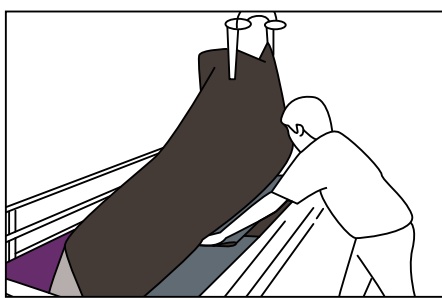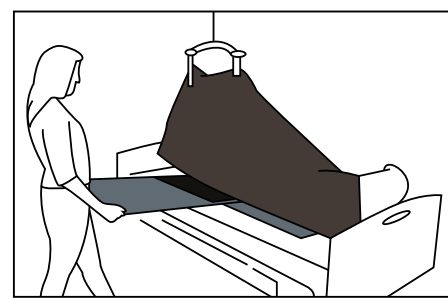
UK Attach the lift to the turn sheet on the side where the sliding flap opening is located. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Insert the sliding flap under the user at the points of least friction (ankle, hollow of the knee, lower back or neck). Slide the straps in so they are pointing forward

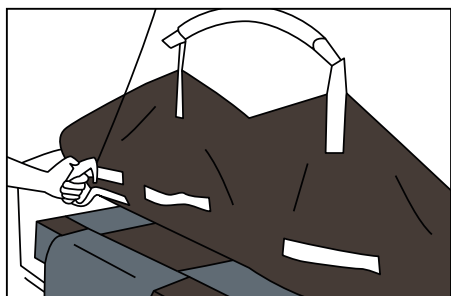
Mount the lift on the opposite side, and raise the side up. Pull out the sliding flap

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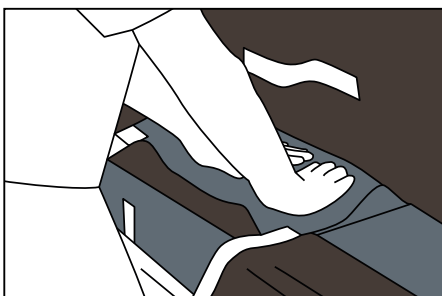
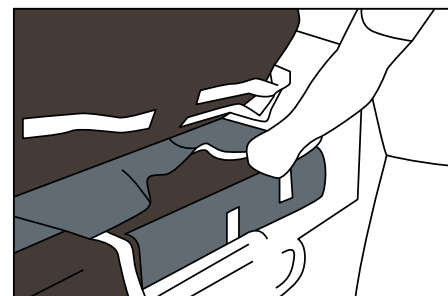
UK Attach the lift to the Master Turner on the opposite side of the sliding flap opening. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Double up the sliding flap and slide it under the user using both hands. Smooth out the sliding flap so it is spread evenly across the cover's non-slip surface. Finish by maneuvering the straps under the user

Then, attach the lift on the opposite side. Raise the lift until the user's shoulder and bottom are not touching the surface beneath. The sliding flap straps should now be visible and can be fastened

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Fördelar / Advantages / Vorteile

Madrascover Exchange 2 / Mattress Cover Exchange 2 / Matratzenbezug Exchange 2

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UK

- Can be fitted while the user is in the bed
- Does not diminish the effects of alternating pressure mattresses
- Works with the mattress
- Possibility of locking the alternating pressure mattress to the bed
- Ensures visibility of the CPR
- Room for alternating pressure mattress air hoses
- Prevents pressure ulcers
- The slide flaps can fix the turn sheet as needed - back and calf

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Madrascover Exchange 3 / Mattress Cover Exchange 3 / Matratzenbezug Exchange 3

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UK

- Can be fitted while the user is in the bed
- Does not diminish the effects of alternating pressure mattresses
- Works with the mattress
- Possibility of locking the alternating pressure mattress to the bed
- Ensures visibility of the CPR
- Room for alternating pressure mattress air hoses
- Prevents pressure ulcers
- The slide flaps can fix the turn sheet as needed - back, pelvis and calf

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Madrascover Exchange ST / Mattress Cover Exchange ST / Matratzenbezug Exchange ST

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UK

- Can be fitted while the user is in the bed
- Does not diminish the effects of alternating pressure mattresses
- Works with the mattress
- Possibility of locking the alternating pressure mattress to the bed
- Ensures visibility of the CPR
- Room for alternating pressure mattress air hoses
- Prevents pressure ulcers

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MC 001-1732 [HMI: 49368]
Master Turner Soft - **Large**

Mått / Dimensions / Größe:
L 215 x B/W 250

MC 001-2540 [HMI: 49370]
Master Turner Two In One Soft - **Large**

Mått / Dimensions / Größe:
L 215 x B/W 145 cm

MC 001-1740 [HMI: 84778]
Master Turner Pro Soft - **Large**

Mått / Dimensions / Größe:
L 215 x B/W 150 cm

MC 001-1700 [HMI: 49207]
Master Turner DUO Soft - Botten / Bottom /
Unterteil - **Large**

Mått / Dimensions / Größe:
L 160 x B/W 150 cm

MC 001-1701 [HMI: 49206]
Master Turner DUO Soft Topp / Top / Oberteil -
Large

Mått / Dimensions / Größe:
L 100 x B/W 150 cm


MC 001-1950 [HMI: 85234]
Master Turner Dragremmar / Locking Straps /
Schließschlaufen

Mått / Dimensions / Größe:
L 47 x B/W 4 cm

MC 001-2500 [HMI: 49873]
Master Turner DUO Two In One Soft Botten /
Bottom / Unterteil - **Large**

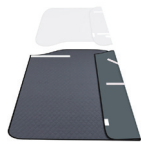Mått / Dimensions / Größe:
L 160 x B/W 145 cm

MC 001-2501 [HMI: 49872]
Master Turner DUO Two In One Soft Topp / Top /
Oberteil - **Large**

Mått / Dimensions / Größe:
L 100 x B/W 145 cm

MC 001-1947 [HMI: 84373]
Master Turner Dragremmar / Pull Straps /
Zugband

Mått / Dimensions / Größe:
L 140 x B/W 5 cm

MC 001-8040 [HMI: 36382]
Master Comfort Stödkil / Support Wedge /
Stützkeile - 2 stk. / 2 pcs. / 2 Stück

Mått / Dimensions / Größe:
L 40 x B/W 15 x H 5 cm

MC 001-8050 [HMI: 34331]
Master Comfort Magkil - Kort /
Stomach Wedge - Short / Bauchkeil - Kurz

Mått / Dimensions / Größe:
L 43 x B/W 25 x H 14 cm

MC 001-8020 [HMI: 45939]
Master Comfort Tvättkil - Medium / Washing
Wedge / Waschkeil

Mått / Dimensions / Größe:
L 70 x B/W 40 x H 30 cm

MC 001-1948 [HMI: 81998]
Master Turner Justeringsremmar /
Adjustment Straps / Regulierungsgurte

Mått / Dimensions / Größe:
L 40-80 x B/W 3 cm

MC 001-1203 [HMI: 34302]
Nylon Master Öppen - Medium / Open / Offen

Mått / Dimensions / Größe:
L 80 x B/W 160 cm

MC 001-1951 [HMI: 81134]
Mattress cover extender - **Large**

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the top piece

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Pernova Hjälpmedel AB