Bruksanvisning / User Guide / Anleitung

Master Turner® Small

○ **MC 001-1712** [HMI: 50302]
Master Turner Soft - **Small**

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○ **MC 001-1917** [HMI: 39113]
Master Turner Two In One Frotté
/ Terry Cloth / Frottee - **Small**

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○ **MC 001-2517** [HMI: 53666]
Master Turner Two In One Soft
- **Small**

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Osäker?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately
- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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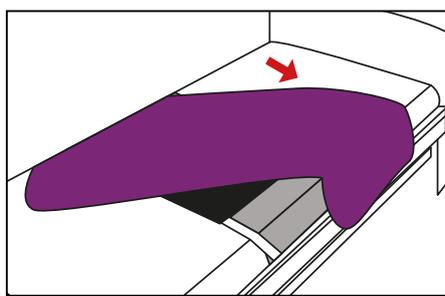
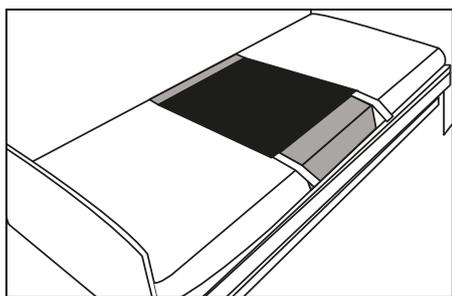
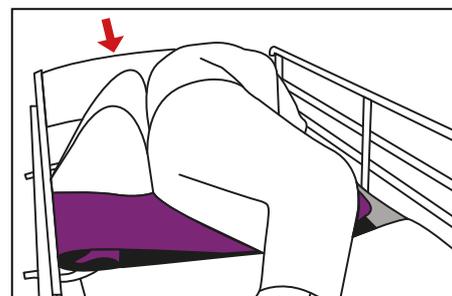
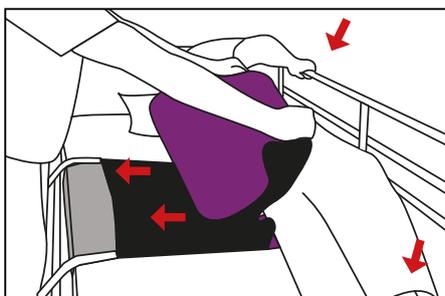
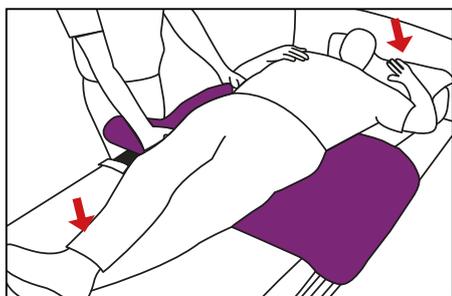
UK You have now placed the selected mattress cover on the mattress

Now place Master Turner Small with the smooth part down against the mattress cover

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UK Cross/bend the user's leg and pull the pillow and shoulder forwards

Help to pull the user towards the edge of the bed while the user grabs hold of the guard rail and pushes away with the crossed/ bent leg

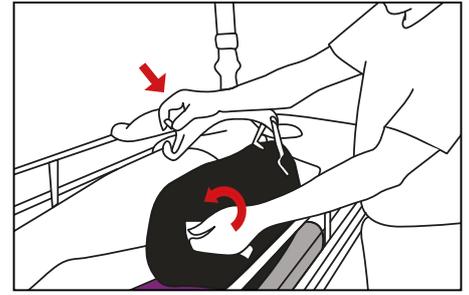
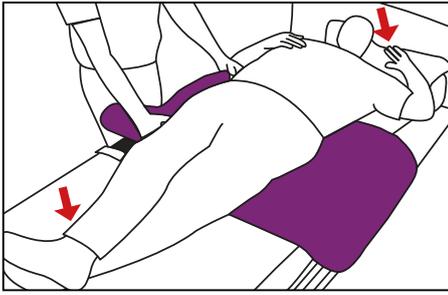
The user is turned onto the side and place the wedge behind the user's back

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UK Cross/bend the user's leg and pull the pillow and shoulder forward

The user is lifted about 20 degrees up from the underlay and help to push the user towards the furthest edge of the bed over towards the guard rail

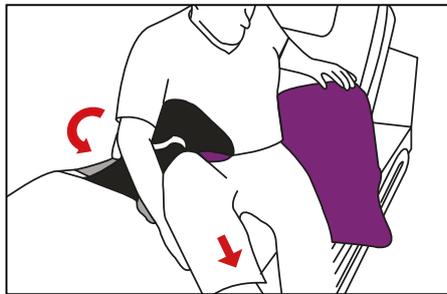
Turn the user onto the side by using Master Turner and lift. The user is positioned and the lift is disconnected from Master Turner

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UK Activate the head section of the bed by the user's knee and elevate the head section

The user can then rotate and move his/her leg over the edge of the bed by using Master Turner

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Fördelar / Advantages / Vorteile

Master Turner Soft, Frotté / Terry Cloth / Frottee, Elastic & Silk - Small

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- UK**
- The padding prevents generation of warmth
 - Can be used with all types of floor or ceiling lift
 - Lever for good handling

- Reduces the risk of pressure sores/shear
- Reduces the reaching distance for the helper and stooped posture

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Master Turner Two In One Soft & Frotté / Terry Cloth / Frottee - Small

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- UK**
- Specially developed for incontinent users
 - The padding prevents generation of warmth
 - Can be used with all types of floor and ceiling lifts

- Lever for good handling
- Reduces the risk of pressure sores/shear
- Reduces the reaching distance for the helper and stooped posture

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps **MUST** be applied to the velcro on the toppiece

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Pernova Hjälpmedel AB