

BioSURE HIV Self Test



WHY TEST FOR HIV?

Testing is the only way to know if you have HIV. It is normal to feel worried, but testing yourself for HIV can be empowering, help put your mind at ease and reduce the anxiety of not knowing. If your test result is positive, don't panic! Treatments for HIV are now so effective that someone diagnosed with HIV can go on to live a healthy, normal life. Testing for HIV can also help ensure that you do not pass the virus on to others.

WHY CHOOSE BioSURE HIV SELF TEST?

The **BioSURE** HIV Self Test delivers results in 15 minutes and only requires a fraction of a drop of blood without any requirement to measure or mix solutions. The first HIV test kit to be **CE** marked means that **BioSURE** has been proven to be easy to use with over **99.7%** accuracy in results. **BioSURE** HIV Self Test is the No1 selling test in the UK.

HOW DOES THE TEST WORK?

The technology behind **BioSURE** is similar to a pregnancy test. Antibodies produced by the body to fight infections are unique, so **BioSURE** works by detecting the antibodies in your blood sample associated with the HIV virus and provides the result according to whether or not the antibodies are detected during the test.

HOW ACCURATE IS THE HIV SELF TEST?

The **BioSURE** HIV Self Test is very effective at diagnosing HIV and proven to be over **99.7%** accurate. If you have been exposed to HIV in the last 3 months, a negative result may not be accurate due to the different times our bodies create antibodies. It is therefore recommended to re-test if you have been at risk.



Don't take our word for it...

Over 1000 reviews and comments from customers says it all really!
Have a read of some anonymous self-testing stories on Pebbl Feedback

For more information visit: www.hivselftest.co.uk

HOW TO TEST:



1

You only need the tiniest drop of blood. The simple paediatric lancet only makes a very small cut in your finger (you might not even feel it!)



2

The test collects exactly the right amount by simply touching the tip onto the drop of blood.



3

Push the tip of the test to the bottom of the pre-measured buffer pot. That's it! You get your easy to read result in just 15 minutes!

After 3 minutes check there is colour running up your test strip. If it isn't, push your device down hard again. Now wait 15 minutes then lay your test within the space in your box to read the results.

